
Universal VESA Bracket

For the BalanceBox® 400 II, the BalanceBox® 650 II and accessories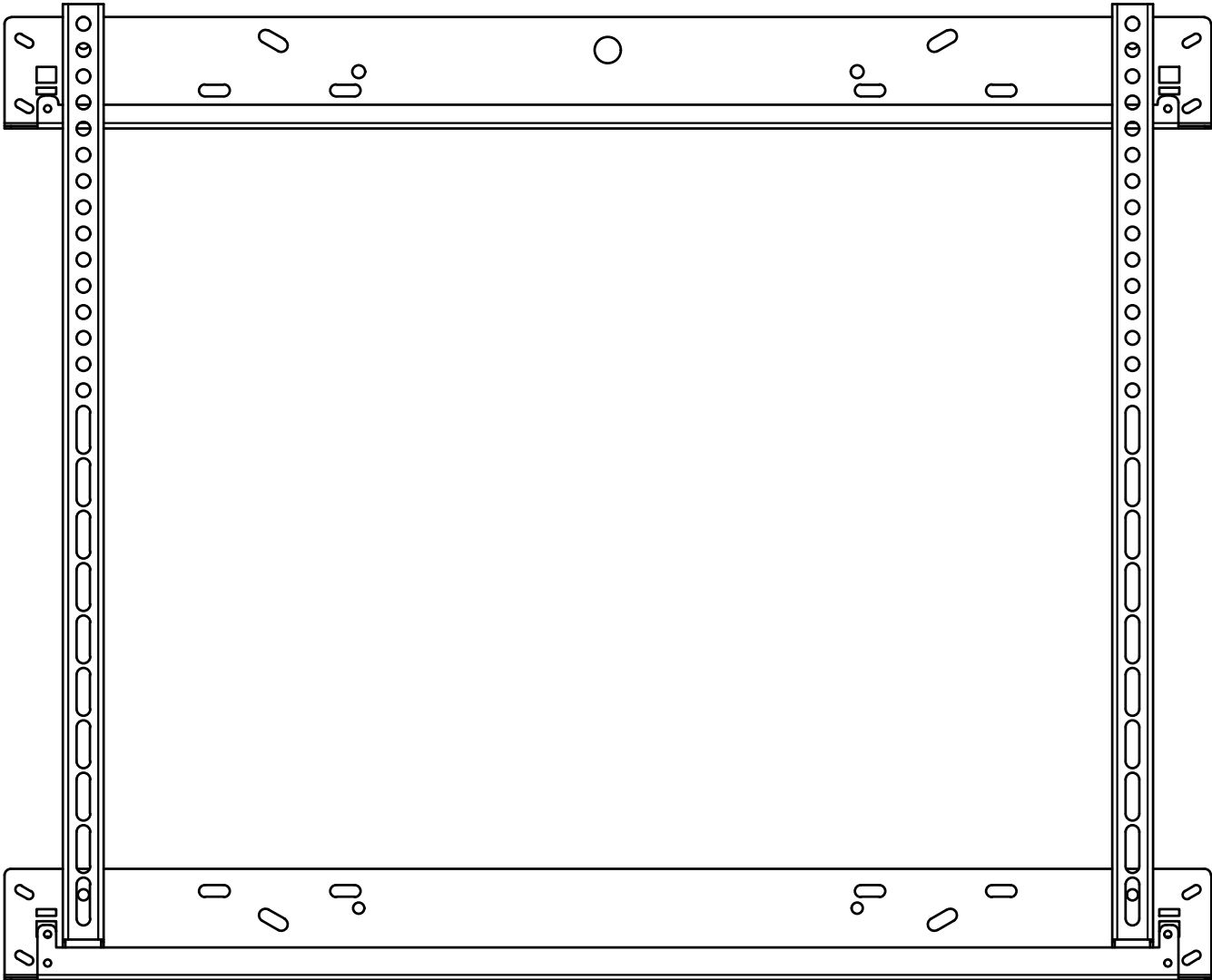

© COPYRIGHT – document number: 902916-06 – note: product specifications may change without prior notice.

- EN
- NL
- DE
- FR
- IT
- ES
- DK
- SV
- NO
- FI
- PL
- AR

Installation video

US

EU

Universal VESA Bracket

General Warnings

Read and understand this manual and supplied safety documentation

Not to be serviced by users

When the symbols below are used in the assembly steps they signify the following:






The step needs to be done by two people



Risk specified by the supplementary sign/note

Tools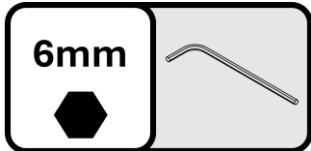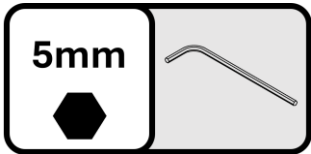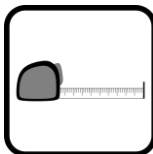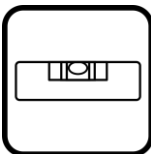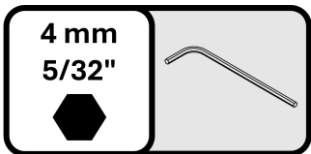


Contents of the box

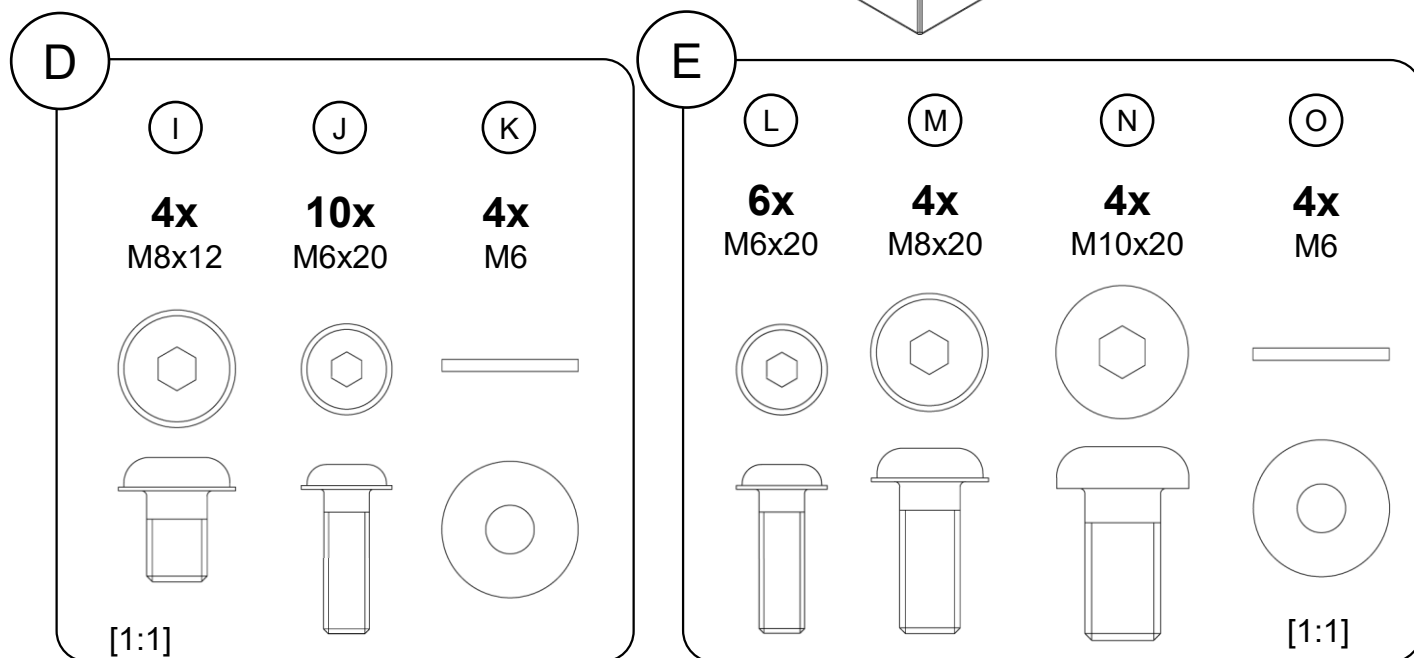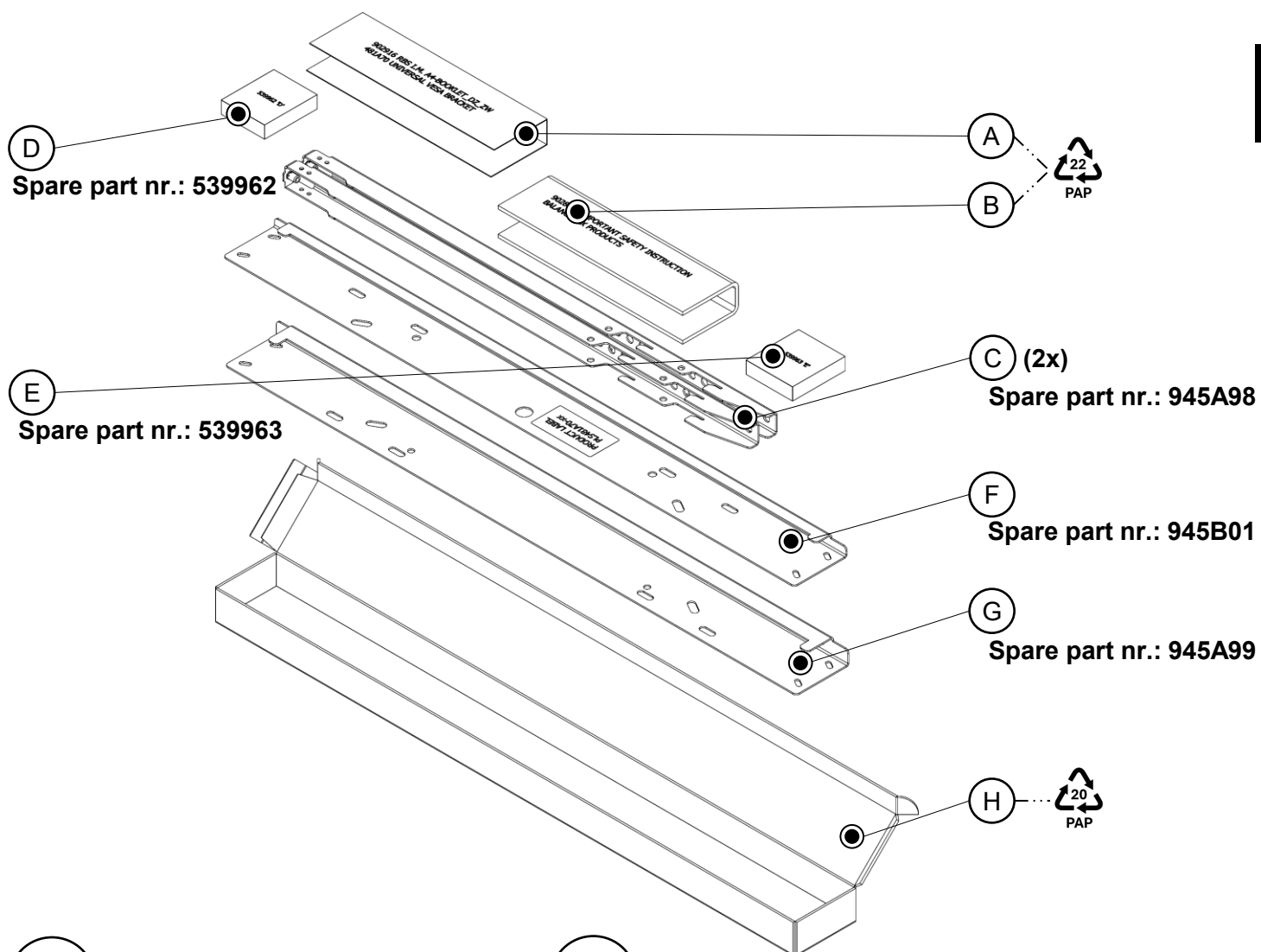
Universal VESA Bracket

1 Choose the configuration of the Universal VESA Bracket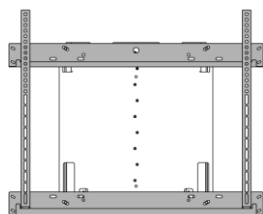

A

Low configuration
Shown example: BalanceBox® 400
Go to step 2 (page 5)


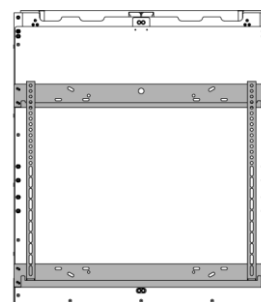


B

High configuration
Shown example: BalanceBox® 650-II
Go to step 3 (page 6)

More specific assembly information can be found in the manual provided with the application:

- A. BalanceBox® 400
- B. BalanceBox® 650-II
- C. Floor Support II for BalanceBox® 400
- D. Mobile Stand Mix
- E. e·Box II Mobile Stand
- F. e·Box II Wall Mount
- G. e·Box IV Mobile motorized height adjustable mount
- H. e·Box IV Motorized height adjustable wall mount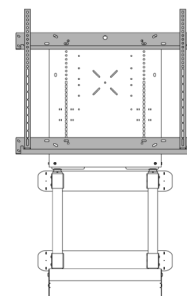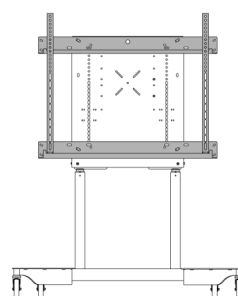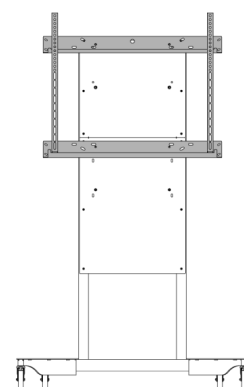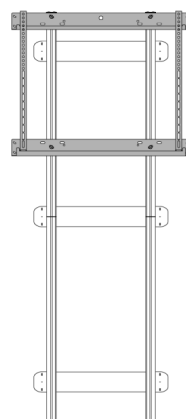

EU




US

Scan the QR code above to view the manuals

2 Universal VESA Bracket in Low configuration (example shown: BalanceBox® 400)

EN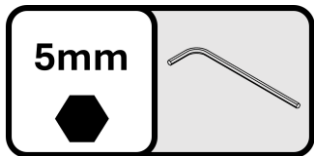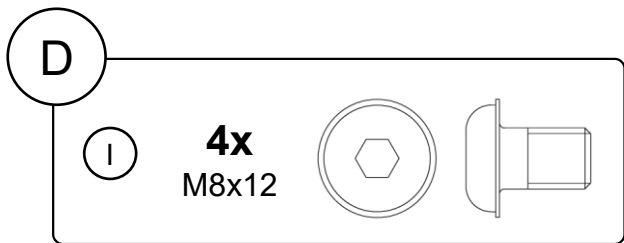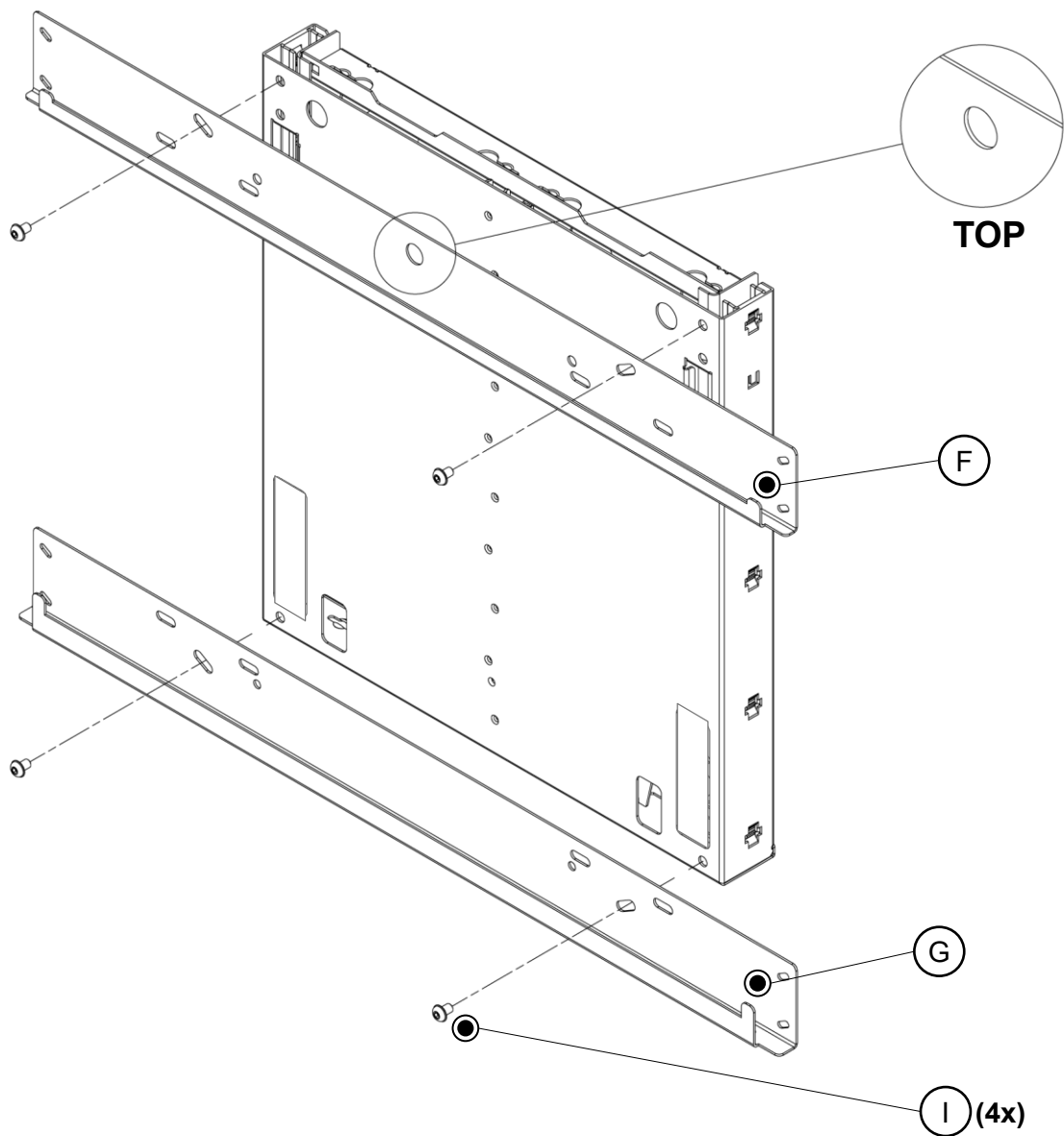


Go to step 4; page 7

Universal VESA Bracket

3 Universal VESA Bracket in high configuration (example shown: BalanceBox® 650-II)

EN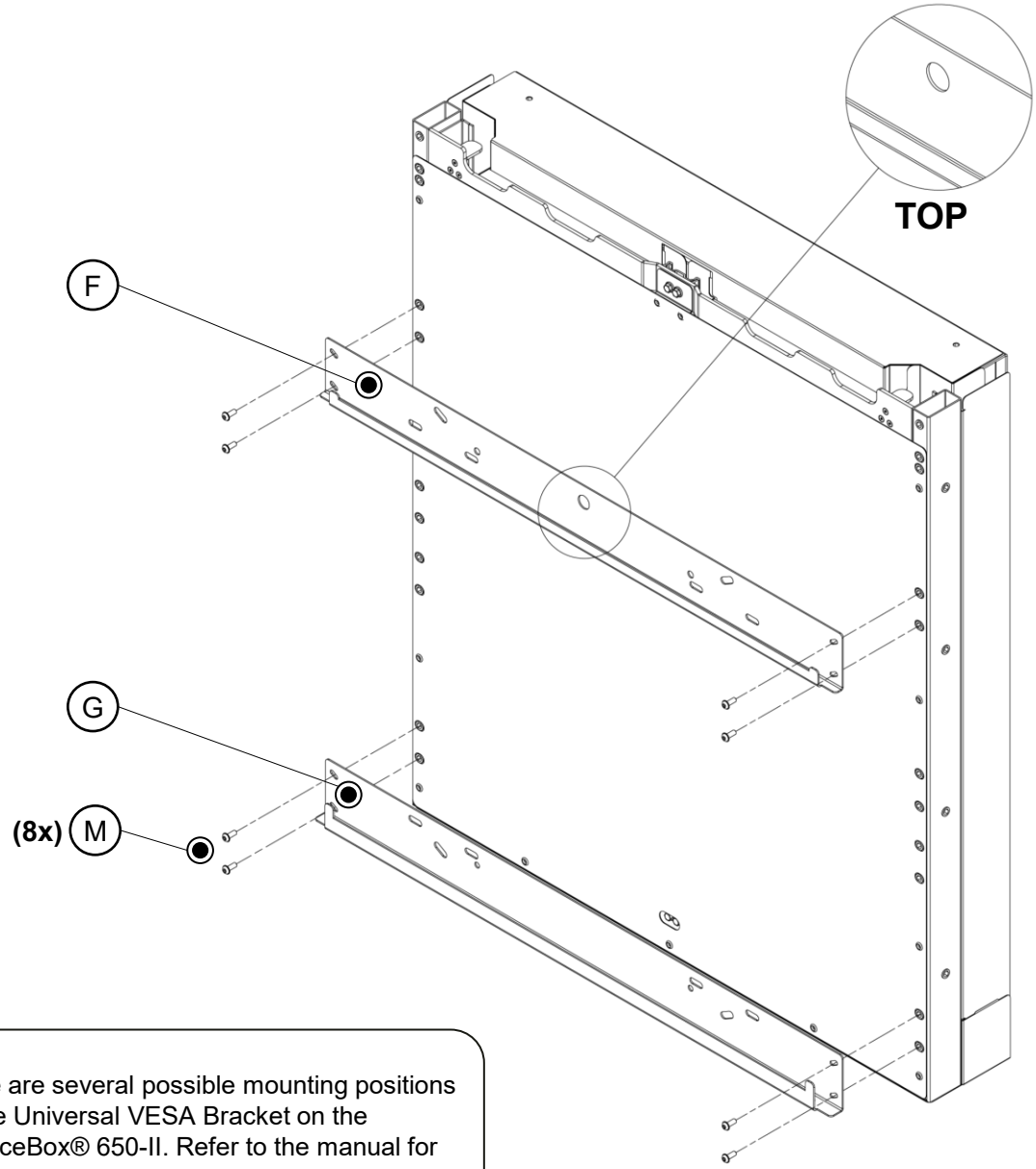

There are several possible mounting positions for the Universal VESA Bracket on the BalanceBox® 650-II. Refer to the manual for details

document number: 902916-06


D

J

8x

M6x20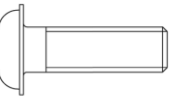

4 mm

5/32"



4 Panel preparation





Refer to manual supplied with the panel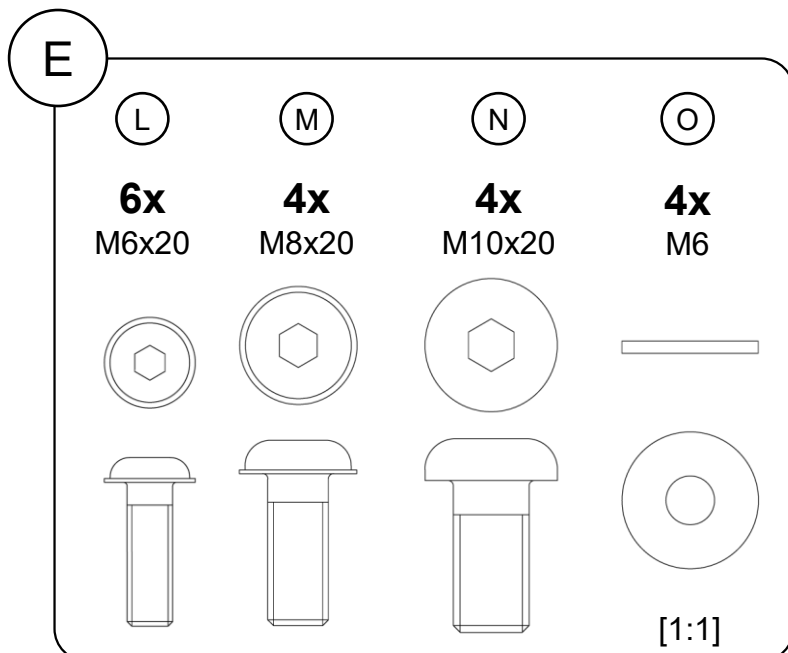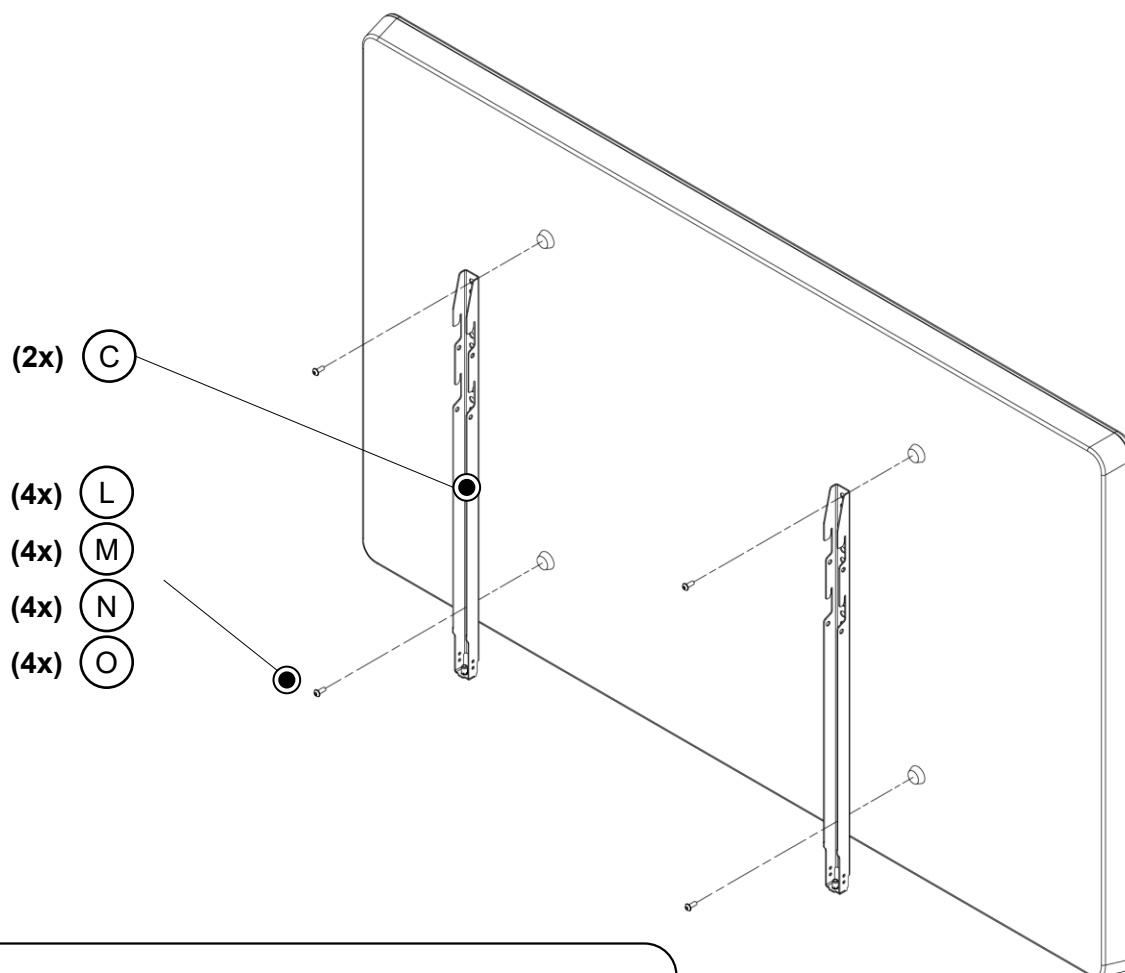


Universal VESA Bracket

5a Panel attachment on the Universal VESA Bracket in **low configuration** (e.g. BalanceBox® 400)

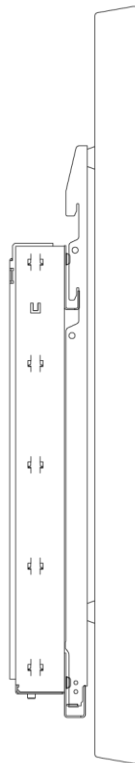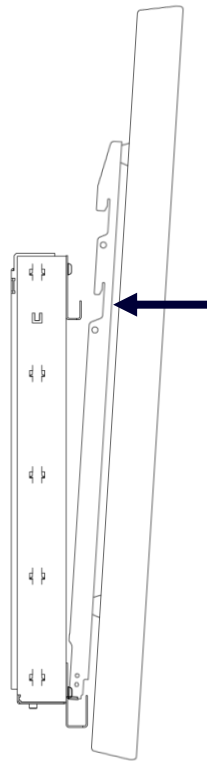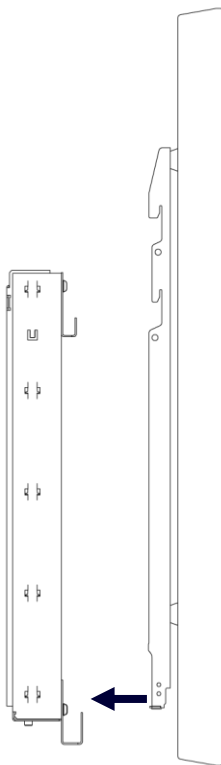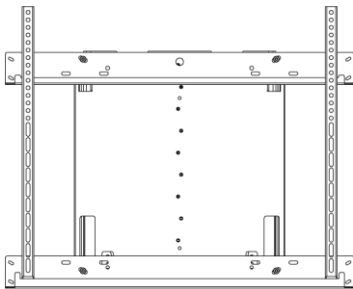
6 Secure the panel on the Universal VESA Bracket using both safety screws **L**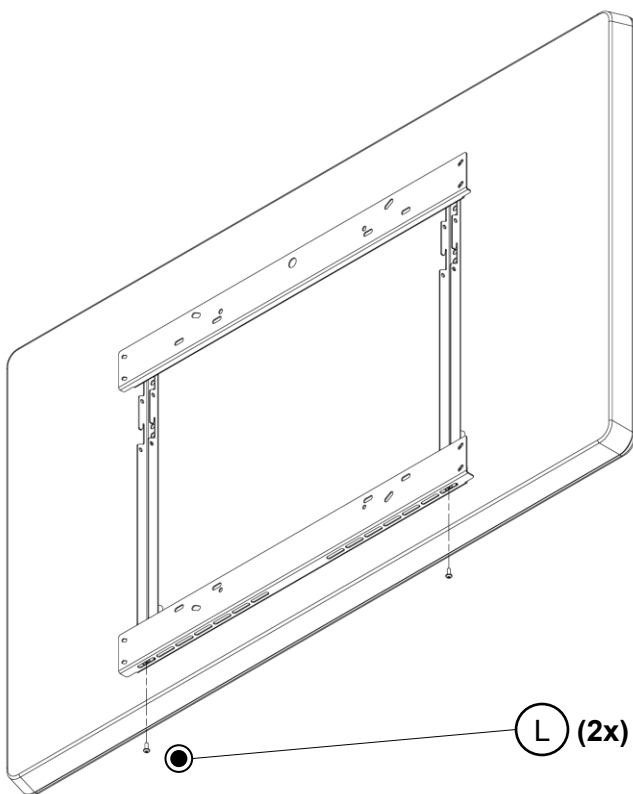

Do not skip this step;
Risk of panel coming loose


4 mm
5/32"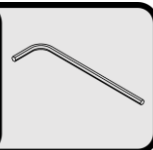



E

L


2x
M6x12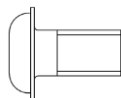

5b Panel attachment on the Universal VESA Bracket in **high configuration** (e.g. BalanceBox® 650-II)

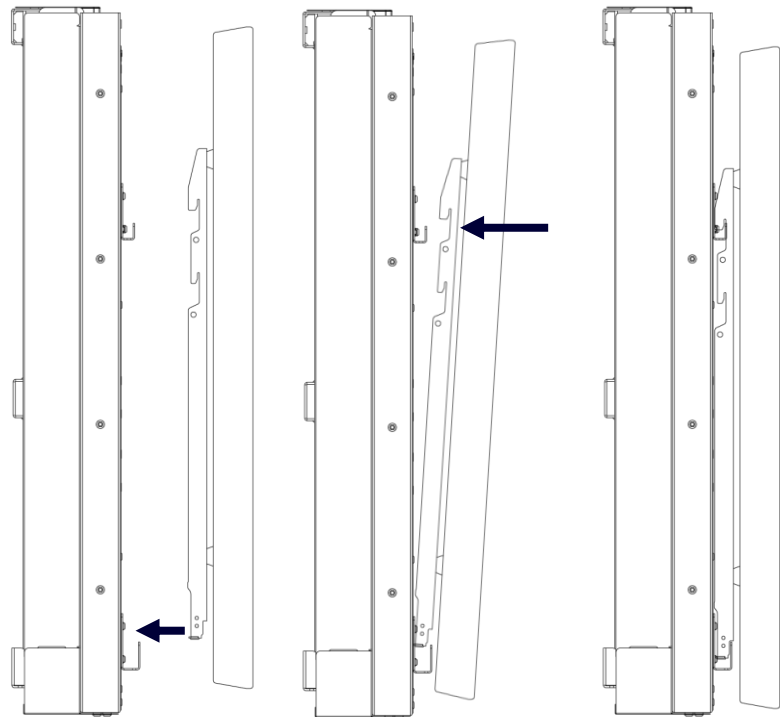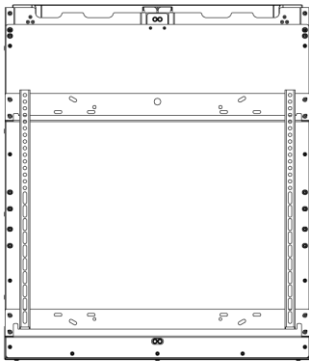
6 Secure the panel on the Universal VESA Bracket using both safety screws **L**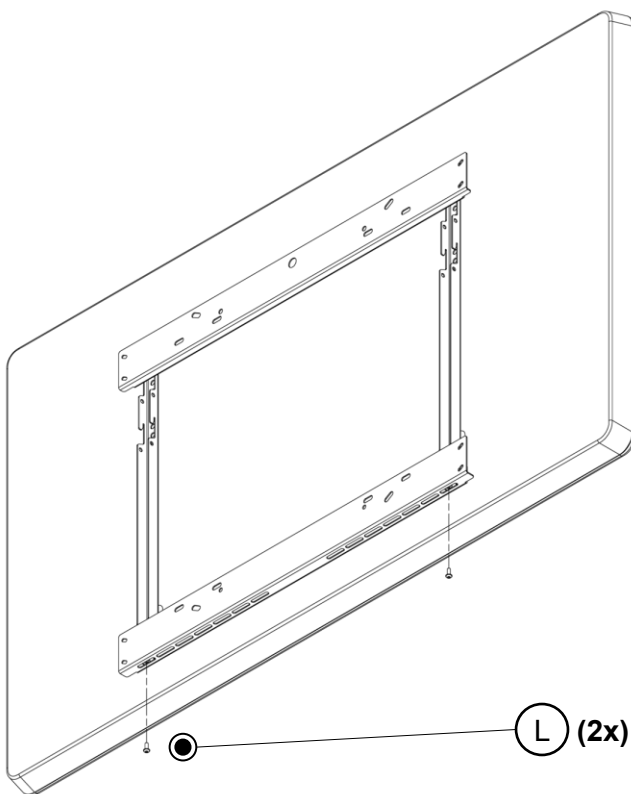

**Do not skip this step;
Risk of panel coming loose**


**4 mm
5/32"**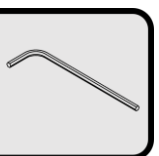

E

L

**2x
M6x12**






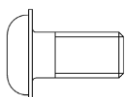
Universal VESA Bracket

7 Continue the assembly steps

If applicable; follow the remaining assembly steps in the manual of the product this has been installed on.

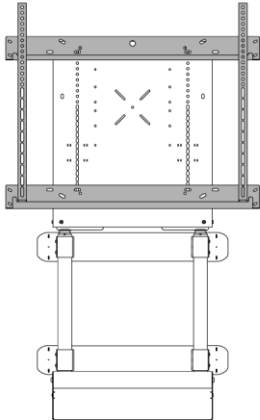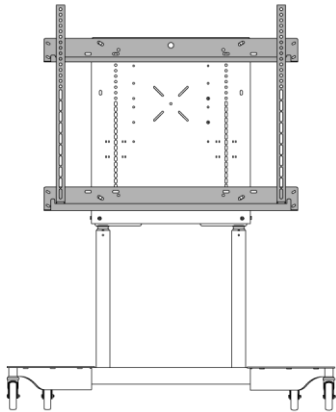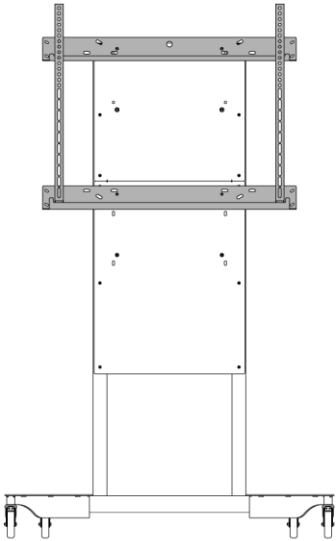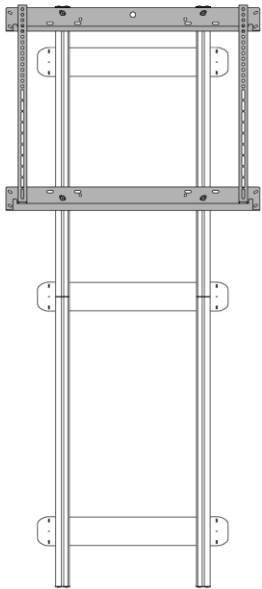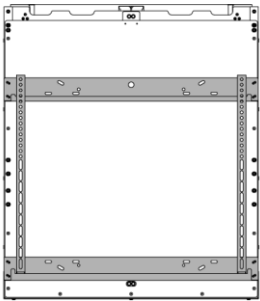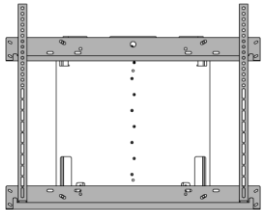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Regout Balance Systems B.V.
Thomas Regout Inc.